Capo Bond

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions Capo Bond

PART 1: introduction
Capo Bond is a light curing, one part primer and bonding system which does not require additional mixing.

Bonding is carried out in a single stage.

The strong bond with enamel and dentin is based upon the same principles that apply to glass ionomer cements. A proper and permanent bond as well as a good biocompatibility are obtained by means of a methacrylate carboxylic acid ester. The material is not to be used in case of irritation or an allergy against methacrylate.

Capo Bond is compatible with all commonly available brands of light curing composite. As Capo Bond is based on ethanol it is hydrophilic and it can be applied to slightly moist dentin surfaces using the "Wet Bonding" technique.

In cases where a dual-curing or self-curing primer and bonding system is recommended, Capo Bond can be dual-cured using a 1:1 mixture of Capo Bond and Capo Bond Activator. Capo Bond Activator is available separately. This special type of application is described in the Capo Bond Activator instructions.

1.1 Composition

Ethanol, methacrylate carboxylic acid ester, photo initiators

1.2 Indications

- Primer respectively Bonder for a tight bond of filling materials to natural hard tooth substances (enamel and dentin).
- Primer respectively Bonder for the fixation of indirect restorations e.g. ceramic inlays, composít inlays, onlays, veneers, crowns and bridges, on natural hard tooth substance with a chemically curing or dual-curing luting composite.
- Capo Bond is well-suited to the impregnation of root canals before filling them respectively before the fixation of root pins or screws.
- Bonder for the repair of crowns and bridges (on precious and non-precious metals as well as on ceramics).

1.3 Contraindications

- Do not use in case of a known allergy against one of the components.

PART 2: Using Capo Bond for filling materials

2. Preparation for light curing composite and compomer filling materials

2.1 Isolation
It is advisable to use a rubber dam.

2.2 Cavity preparation

Before beginning preparation, clean the tooth with pumice and water. To ensure that as little healthy tooth structure as possible is reduced, only prepare the tooth minimally. Enamel cavity margins must be bevelled slightly (0.5–1.0 mm) to enlarge the adhesive surface in the enamel and increase the bond strength.

2.3 Pulp protection

In very deep cavities those areas in close proximity to the pulp must be coated with a thin layer of calcium hydroxide cavity liner.

2.4 Etching the enamel

The Total-Etch technique is recommended: Apply Capo Etch to enamel and dentin surfaces according to the manufacturer’s instructions. Next, rinse with water for 20 seconds and slightly dry with water-free and oil-free compressed air. The dentin must not dry out completely, as a slightly moist dentin surface is essential for the function of Capo Bond. After treatment, the etched enamel surface appears clearly chalky-white. The etched enamel surface must not be touched or contaminated with saliva prior to applying the Capo Bond. Should it become contaminated, repeat the procedure, i. e. etch, rinse with water and dry again.

Warning:

Avoid contact of etching gel with mucous membranes, eyes and skin. In case of an accidental contact, rinse with copious amounts of water.

For typical applications of compomer material, etching of the enamel is usually not necessary, but enamel etching considerably improves adhesion to the enamel.

2.5 Application of Capo Bond
Shake Capo Bond before use. Apply Capo Bond in generous amounts to the dentin and enamel surfaces with a brush and brush intensively for 30 seconds. Then dry carefully with oil-free compressed air for about 15 seconds. Cure the Capo Bond coating by irradiating with a dental lamp for 20 seconds before applying a second coat of Capo Bond.

CAUTION: Do not spill Capo Bond! If not used immediately, store Capo Bond in dim light to prevent premature polymerization by incident light. Capo Bond will not cure spontaneously.

2.6 Applying the second coat of Capo Bond
Use a brush to apply a second coat of Capo Bond to the surfaces being adhered and brush it in vigorously for approximately 30 seconds. Then dry for approximately 15 seconds with oil-free compressed air and cure for 20 seconds with a dental lamp. The light curing composite filling can then be placed.

2.7 Placing, curing and finishing the filling
The Capo Hybrid instructions describe how the composite filling is placed, cured and finished.

Priming the root canals

2.8 Preparation
The use of a rubber dam to isolate the teeth is strongly recommended. Clean and prepare the teeth with hydrogen peroxide solution (3 %) or sodium hypochlorite solution. When placing passive or threaded endodontic posts with luting composites, refer to the manufacturer’s instructions. Prepare the canal to determine the length of the root post required. Adapt the post to the canal and reduce the length if necessary. Remove the post, keep it in a safe place and prepare the canal for cementing the post.

2.9 Priming the root canals
As the surfaces of the root canal cannot be reached with a brush, Capo Bond must be injected into the prepared canals with a syringe to rinse them for 60 seconds in four separate stages. Ensure that all surfaces being bonded are wetted continually. Wherever possible, use a brush to work the Capo Bond in and enhance wetting. Soak up excess Capo Bond from the root canal using, for example, a soft paper point and dry the treated surfaces carefully for 15 seconds with oil-free compressed air. The Capo Bond is then cured for 20 seconds with a dental lamp.

2.10 Applying the luting composite
Place and cure the luting composite as described in its manufacturer’s instructions.

PART 3: Special applications for Capo Bond

3. Using Capo Bond on precious/non-precious metals and porcelain for repairing crown and bridgework

3.1 Conditioning the surfaces of the restoration
Roughen the surfaces of the metal or fractured porcelain with a diamond instrument (40 µm) – if necessary, sandblast them in addition (with an intraoral sandblaster). Apply Capo Etch to the rough surfaces, rinse them with cop

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