# PaintAcryl

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Instructions

### Indication

Building up patterns of outer copings with a brush

### Contraindication:

If the user is known to be allergic to one of the constituents of this material, it must not be used.

### Side-effects:

None known to date.

### Properties:

- Burns out completely
- Minimal shrinkage

### Processing:

- Dispense the required amounts of liquid and powder into separate mixing vessels.
- Dip the tip of the brush into the liquid and then pick up a small amount of powder.
- The resin swells on the brush tip and can

be placed and shaped exactly on the coping.

- Repeat this procedure several times until the outer coping has been built up.
- Carefully raise the pattern from the inner coping and check the marginal fit as well as the inside of the pattern.
- Replace the pattern on the inner coping and use a cutter to trim it to uniform thickness.

### Please note:

- If this material is being used on a plaster model, FuturaSep Plus separating agent should be applied.
- Clean the brush with PaintAcryl liquid after use or, if necessary, during use.
- Do not pour any leftover liquid or powder back into the bottle.

### Colour:

red

**Storage:**

Store at 10 - 25 °C

**Presentation:**

100 g powder

100 ml liquid

1 Set: 100 g powder and 100 ml liquid

**Contains methacrylates**

May cause an allergic skin reaction.

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