Relight

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions for use

Designation of purpose:

The light-curing denture varnish is used to chemically smooth the surface of a denture base after finishing.

General product description:

Light-curing one-component material for sealing the surfaces of denture bases as well as Relight Cleaner and Relight Primer for the preparation of the surfaces. For use by dental technicians in a dental lab.

Composition:

Relight Cleaner: Methyl ethyletone
Relight Primer: Methyl methacrylate
Relight Lacquer: Methyl methacrylate (MMA), urethane acrylate, initiators, stabilizers

Indications:

- for the surface treatment of denture base materials
- for the surface treatment of tray materials
- for the surface treatment of temporary crown and bridge materials
- for the surface treatment of refinished or worn dental restorations

Contraindications:

Do not use if allergy to any of the components is known. In case of allergy or hypersensitivity of the patient to any of the components, do not use this product or use it only under strict supervision of the attending physician / dentist.

Interaction with other agents:

Phenolic substances (such as e.g. Eugenol) inhibit polymerization. Therefore, do not use materials which contain such substances.

Application / Processing:

Roughen the area of the surface to be painted before application or blast with 50 µm aluminum oxide and clean. Remove all sanding dust. Coat the roughened surface with Relight Cleaner and allow to react for 4 min. Next, brush on a thin layer of Relight Primer and allow to react for 4 min. Apply a thin layer of Relight Lacquer and cure.

Troubleshooting / FAQ list:

Error	Cause	Remedy
does not cure	insufficient polymerization	observe polymerization times
		<ul style="list-style-type: none"> – check lamp, replace lamp if necessary – do not use intermediate polymerization
smeary surface	insufficient polymerization	observe polymerization times
		check device / service device regularly
lacquer chips	insufficient bond	surface must be free of sanding dust
		observe processing times

General information:

Use a suitable light polymerization device with an emission spectrum of at least 310 - 500 nm. The required physical properties are only achieved with faultless lamps. Therefore, regular checks of the light intensity according to the manufacturer's specifications are necessary. Do not use intermediate polymerization devices.

Polymerization times:

Light polymerization device	Time
HiLite / UniXS	90 sec.
Labolight LV-II / III	5 min.
Solidilite EX	5 min.

Storage:

Storage temperature 10 - 25 °C / 50 - 77 °F.
Tightly close syringe directly after use.

Shelf-life:

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

Side-effects:

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e. g. allergies) or local discomfort, however, cannot be ruled out completely. All serious incidents occurring in connection with the use of this product must be reported to the specified manufacturer and the relevant competent authority.

Disposal:

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

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