# After Bleaching Care

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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# Instructions for use

## Description:

After Bleaching Care is a water based paste which provides for a reduction of tooth sensitivity after the whitening procedure.

Pleasant peppermint flavor.

## Intended Use:

After Bleaching Care is a paste to be applied onto the teeth for post-treatment after the whitening procedure.

## Composition:

Purified water, Xylitol, glycerin, potassium nitrate, sodium fluoride, thickener, flavoring.

## Indications:

For post-treatment after a tooth whitening procedure.

## Contraindications:

If a patient has known allergies or intolerances towards components of this product, it must not be used or otherwise, it shall be used only under strict supervision of the attending physician/dentist.

## Interactions with other agents:

When using the product, the dentist should consider known cross-reactions and interactions of the product with other materials already in the patient's mouth.

## Application with an individual tray

1. Clean the individual tray (or thermoforming tray) under running water after the whitening procedure.
2. Spread After Bleaching Care over the insides of the tray and apply the tray to upper and lower jaw.
3. Keep the tray in the patient's mouth for about 3 to 5 min.
4. Remove the tray.
5. Ask the patient to rinse well. The patient must not eat or drink for a minimum of 30 minutes following the treatment.
6. Rinse out or brush out any After Bleaching Care residue in the tray.

## Application without a tray

1. If necessary, dab off superfluent saliva from the tooth surfaces.
2. Spread a generous amount of After Bleaching Care on the tooth surfaces with a disposable brush, a protective glove or, in difficult interproximal situations, with an interproximal brush.
3. Leave to act for about 3-5 min.
4. Ask the patient to rinse well. The patient must not eat or drink for a minimum of 30 minutes following the treatment.

## Shelf-life:

The maximum shelf-life is printed on the label of each syringe. Do not use after the expiration date. Store at 3 °C - 25 °C (37.4 °F -77 °F).

## Side effects

Undesirable side effects of this medical device are extremely rare when properly processed and used. Any serious incidents in connection with the use of this product must be reported to the manufacturer indicated below and to the relevant competent authority.

## Note on disposal

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

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