# FuturaSelf

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Instructions for use

### General product description:

Cold-curing acrylic on methylmethacrylate basis.

### Composition:

- Powder: pearl polymerisate composed of polymethyl methacrylate (PMMA), pigments, initiators
- Liquid: methylmethacrylate, tetramethylene dimethacrylate, initiators, stabilizers
- FuturaSelf is free of cadmium

### Indications:

Repairs, extensions and relinings of existing restorations in the flow method.

### Contra-indications / interactions with other agents:

Do not use in case of a known allergy against one of the components.

### Application / processing:

#### Preparatory steps:

Produce plaster model and wax-up as usual. The prefabricated teeth set up in the wax are fixed with plaster or silicone (stent / flask). Subsequently, the wax is eliminated as usual. Roughen the acrylic teeth, provide them with undercuts and coat them with monomer or HS Cross-Liquid (Schütz Dental) to ensure that they are securely retained. The surfaces of the plaster models are insulated with the alginate separating agent (FuturaSep Plus, Schütz Dental).

#### Mixing ratio:

5 weight parts powder, 3 weight parts liquid. Veins can be added, if desired.

### Repairs, extensions and relining of existing acrylic appliances in the flow technique:

Mix powder and liquid (according to the mixing ratio) and stir with a spatula to produce a homogeneous mixture.

The acrylic dough is then flowable and dripable and can be processed immediately. Drip the flowable dough onto the roughened surface and spread evenly. After polymerization the restoration can be ground and polished immediately.

As soon as the dough reaches its plastic phase, it must not be touched or manipulated any longer.

Apply larger amounts of FuturaSelf in layers to avoid the formation of pores. A safe and bubble-free polymerization is only guaranteed under pressure. The working range of the product is 5 min.

The polymerization time is 15 min. in a pressure polymerization unit, at a pressure of 2–5 bar and at a water temperature of 45 °C (113 °F).

### Finishing the material:

After polymerization, the restoration is lifted from the model and it is finished with suitable tools.

Silicone polishers (wheel, lentil, roll) as well as hard metal cutters or diamond grinding tools are suited for finishing the material.

Prepolish with sandpaper or pumice.

### Polishing:

Polish the material with goat hair brushes, polishing paste and soft woolen wheels. A careful surface treatment and polish are a precondition for an optimal result.

**Troubleshooting / FAQ list:**

<b>Problem</b>	<b>Cause</b>	<b>Remedy</b>
acrylic does not cure properly	Incorrect mixing ratio	<ul style="list-style-type: none"> <li>– observe instructions for use</li> <li>– observe mixing ratio</li> </ul>
acrylic shows a whitish tint along the bottom	<ul style="list-style-type: none"> <li>– insufficient insulation</li> <li>– model was not soaked in water</li> </ul>	<ul style="list-style-type: none"> <li>– insulate sufficiently</li> <li>– check insulation agent, if necessary</li> <li>– soak model in lukewarm water for ca. 10 min.</li> </ul>
acrylic forms bubbles along the bottom	model was not soaked in water	soak model in lukewarm water for ca. 10 min.
acrylic surface white / spongy	too much time passed until the acrylic was injected or until the acrylic was put in the pressure pot	observe processing times
color deviations of repairs	incorrect mixing ratio	observe mixing ratio
acrylic fractures when lifted	improperly insulated	check insulation
acrylic / prosthesis shows whitish streaks	<ul style="list-style-type: none"> <li>– improperly mixed</li> <li>– powder content too high</li> </ul>	<ul style="list-style-type: none"> <li>– mix acrylic dough thoroughly with a spatula</li> <li>– observe mixing ratio</li> </ul>
acrylic polymerizes too quickly	<ul style="list-style-type: none"> <li>– temperatures too high</li> <li>– work steps too long</li> </ul>	<ul style="list-style-type: none"> <li>– observe temperatures</li> <li>– observe working range</li> </ul>
splint fractures	design too thin / unsuitable	ensure a sufficient layer thickness (min. 2–3 mm)

**General notes:**

For deviations from 21 °C room temperature: Higher temperatures reduce the above times, lower temperatures increase them.

To further reduce the risk of intolerance reactions, we recommend storing the restoration in lukewarm water for at least 12 hours before insertion.

**Storage:**

Storage temperature at 10–25 °C / 50–77 °F. Close container carefully.

**Shelf-life:**

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date.

**Side-effects:**

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e. g. allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is

doubtful that the side-effect has been caused by our product – please kindly contact us.

**Contra-indications / interactions:**

If a patient has known hyper-sensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, the composition of the medical device we supply is available upon request. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

**Note:**

Please supply the attending physician with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

**Disposal:**

Powder and liquid must be disposed of as hazardous waste.

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