## Tizian Flow Resin

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#### Produktbeschreibung

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### Instructions for use

#### Product description

Tizian Flow Resin is a light-curing synthetic material for modelling crowns or bridges milled using Tizian Mill.

Hardening may be carried out using a normal polymerisation lamp with a wave length range of 350 - 550 nm.

#### Use

Modelling of crowns, bridges or inlays.

#### Application

Block out any sharp edges in the incisor area and the area below with wax. Then insulate the plaster stump with vaseline.

Remove cap and affix an application needle. Tizian Flow Resin is applied directly from the syringe.

Using a canula, apply Tizian Flow Resin 1 mm above the preparation border, on the stump and harden for 15 seconds using a Tizian-Q-PLC polymerisation lamp.

The coating should not be less than 0.7 mm thick, following preparation.

After polymerisation, take the cap down from the stump and polymerise again from the inside.

Place the cap back on the stump and mould the edge with Tizian Flow Resin.

Finally, where necessary, sand the external form. Be careful, however, not to disturb the edge.

In order to ensure sufficient polymerisation, layers must not be greater than 2 mm. For larger interventions, polymerisation must be carried out between layers.

We recommend a polymerisation time of 15 seconds per layer.

Since it doesn't burn without leaving a residue, Tizian Flow Resin may not be used for embedding and casting.

#### Storage

Store the material in a dark place at a temperature of 10 - 25 °C.

#### Best before date

A best before date is printed on the label of every syringe. Please do not use this product once the date has passed.

#### Composition

BisGMA, Bis-methacrylate, pyrogene silica, photo-initiators, pigments

Colour: blue

#### Disposal

In accordance with local and / or statutory regulations.

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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