Powder:

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Powder contain cristobalite and dibenzoyl peroxide

Danger. May cause an allergic skin reaction. Causes damage to the lung through prolonged or repeated exposure. Route of exposure: Inhalation. Do not breathe dust / fume. Wear protective gloves. Take off contaminated clothing and wash it before reuse. If skin irritation or rash occurs: Get medical advice / attention.

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Contains methyl methacrylate, diphenyl(2,4,6-trimethylbenzoyl) phosphine oxide, aromatic urethane acrylate

Danger. May cause an allergic skin reaction. Avoid breathing vapours / spray. Wear protective gloves / protective clothing / eye protection / face protection. If skin irritation or rash occurs: Get medical advice / attention.

Contient méthacrylate de méthyle, diphenyl(2,4,6-trimethylbenzoyl)-phosphine oxide, acrylate de urethane aromatique

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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