# Tizian Blanks PMMA pink

CE 0297

**SCHÜTZ DENTAL**  
Micerium Group

Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany  
Tel.: +49 (0) 6003 814-0 · Fax: +49 (0) 6003 814-906  
info@schuetz-dental.de · www.schuetz-dental.de

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Instructions for use

### For the production of denture bases in dental milling systems:

Acrylic blanks made of PMMA (polymethyl methacrylate).

### Indications:

Tizian PMMA pink Blanks are made of a millable acrylic (PMMA). They are indicated for the production of denture bases. The material can be individualized with conventional denture colors.

### Finishing / polishing:

Finish with cross-cut hard metal mills. Polish with conventional high lustre polishing paste and goat bristle brushes.

**Important: Do not overheat the acrylic when using mills, steam clean, or similar tools to finish it.**

### Processing:

The optimal ambient temperature for milling Tizian PMMA pink Blanks is 20–25 °C (68–77 °F). Please always follow the instructions supplied with your milling machine.

The milling parameters strongly depend upon the milling machine, the CAM software as well as on the milling strategies used. In case of doubt, it is always best to use the parameters for PMMA.

Drill holes are prepared in accordance with the measurements of the acrylic teeth which will be used. The teeth, which were previously roughened and prepared with an undercut, are covered with a thin layer of HS Cross Liquid. Avoid pooling and leave the prepared teeth to dry for 2–3 minutes. Position the teeth in the prosthesis and thinly cover the contact area with PMMA (FuturaJet). Polymerize in a pressure pot at 2–4 bar and at 45 °C (113 °F), for 15 minutes.

### Please note!

When milling Tizian PMMA pink Blanks, respirable dust can be released which may damage the lungs or lead to irritations of the eyes and skin. Therefore, the material must only be processed while using a proper and efficient extractor system.

Please adhere to the following safety rules:

- Only process PMMA blanks underneath a local extractor system.
- Ensure that product dust will not come in touch with eyes or mucous membranes.
- Do not inhale dust.
- Wear face mask and protective goggles.

### Storage:

Store in a dry place at temperatures between 10 °C and 25 °C (50 °F and 77 °F). Protect from direct sunlight and from humidity.

### Shelf-life:

The maximum shelf-life is printed on the label of each pack. Do not use after the expiry date.

### Disposal:

The material can be disposed of with normal household trash.

### Guarantee:

Our processing instructions, regardless of whether they are issued verbally, in writing or in the form of a practical demonstration, are based upon our own experience and can therefore be regarded solely as guidelines.

### Physical properties:

The physical properties are mean values and may slightly vary between delivery batches.

Composition

PMMA	> 98%
anorganic pigments	< 2%

Vicker's hardness	137 MPa
bending strength	91 MPa
E-module	2650 MPa
water absorption	26.8 µg/mm <sup>3</sup>
water solubility	1.6 µg/mm <sup>3</sup>
residual monomer	< 1,0 %
shelf life	10 years

**Side effects**

Undesirable side effects of this medical device are to be expected only in extremely rare cases if the product is handled and used correctly. However, immune reactions (e.g. allergies) or local discomfort cannot be excluded completely. If you become aware of undesirable side effects, please notify us, even in case of doubt.

**Contra-indications / interactions**

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, we will supply the composition of our medical device upon request. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

**Note**

Please ensure that all above information is readily available to the attending dentist in case this product is used. Please also note the material safety data sheets available.