Bleach'n Smile Dental Dam

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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Instructions for use

Bleach'n Smile Dental Dam is a light-curing gingiva guard which is used similar to a rubber dam for isolating the gingiva from the teeth requiring treatment.

Designation of purpose:

Light-curing material for covering/insulating the gingiva of the teeth to be treated during in-office bleaching.

Composition:

Aliphatic di-urethane dimethacrylate, tetramethylene dimethacrylate, silanized glass powder, light initiators, stabilizers, color pigments

Indications:

For covering the gingiva during bleaching in the dental practice.

Contraindications:

In case of allergies or intolerances against one of the components, the material must not be used at all or only under strict supervision of the attending physician / dentist.

Hazard note:

Contains di-urethane dimethacrylate, tetramethylene dimethacrylate. May cause allergic skin reactions. Toxic to aquatic life with long lasting effects.

Safety advice:

Avoid breathing vapours/spray. Avoid release to the environment. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.

Cross-reactions with other agents:

The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

Preparation:

Take appropriate measures (e. g. use cheek and lip retractors) to ensure that the tongue and lips are kept in a safe position during treatment (and the subsequent bleaching procedure) and do not contact the material. Remove the seal of the syringe and replace it with the syringe application tip. Regulate the flow of the material.

Application:

Dry the gingiva. Apply Bleach'n Smile Dental Dam to a width of approx. 4 - 6 mm and a thickness of 2 mm along the area to be protected and press the material into the interdental spaces. (Foto 1) Cure for 20 to 40 sec. using a standard light-curing hand lamp. Do not hold the lamp too long on one area. After curing, remove any excess material on the teeth using a scalpel. The bleaching gel can now be applied. (Foto 2)

Finishing:

After bleaching, remove the Bleach'n Smile Dental Dam by lifting the material at one end with an instrument, such as a probe, and pull it off. Remove the syringe application tip and replace it with the seal. (Foto 3)

Shelf life and storage:

The maximum shelf-life is printed on the label of each packaging. Do not use after the expiration date. Bleach'n Smile Dental Dam should not be used after the expiry date. Store Bleach'n Smile Dental Dam in a dark place. Avoid heat and direct sunlight. Close container carefully after use. Store at 3 - 25 °C.

Side effects::

Adverse reactions to this medical device are extremely rare when properly processed and used. Any serious incidents arising in connection with the use of this product must be reported to the manufacturer indicated below and to the relevant competent authority.

Note on disposal:

Residual quantities and packaging material must be disposed of in accordance with local and/or statutory regulations.

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Schütz Dental GmbH