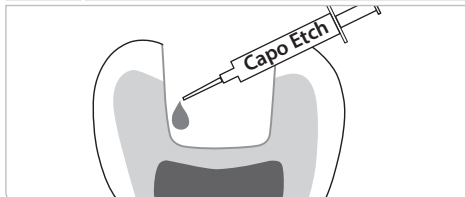
Capo Bulk Fill / Capo Bulk Fill plus Composite – Step by Step Anleitung

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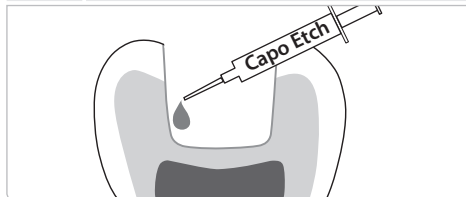
www.schuetz-dental.de • info@schuetz-dental.de

Mandler 10 / 2023 – 33.000

Capo Bulk Fill /Capo Bulk Fill plus Composite – Step by Step Instructions


Conventional preparatory treatment of the tooth substance: Cavity preparation according to the principles of adhesive technology.

1 Adhesive system

Apply Capo Etch etching gel to the preparation surface. Etch enamel for 30 sec. and/or dentin for 15 sec. Subsequently, rinse well with water. Dry the etched surface with oil-free compressed air.

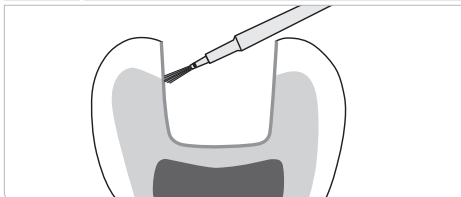2 Application of Capo Bond


Rub in intensely with a brush for 30 sec.

3 Drying

Dry with oil-free air for approx. 15 sec.

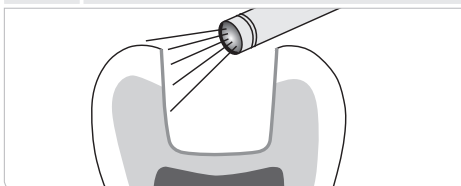4 Polymerization

Polymerize the bonding layer with a polymerization lamp for 20 sec. Perform steps 2 to 4 a second time.

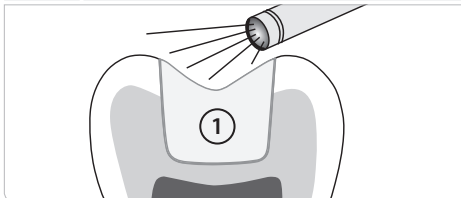
5 Application of Capo Bulk Fill / Capo Bulk Fill plus Composite

Optionally, use Capo Bulk Fill /Capo Bulk Fill plus Composite as a cavity liner.


Apply Capo Bulk Fill /Capo Bulk Fill plus Composite ① in layers of up to 4 mm directly into the cavity. The application of a cover layer with e.g. Capo Natural is optional for class I cavities.

6 Polymerization

Polymerize each layer for 20 sec.

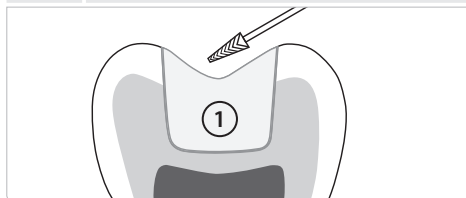7 Finishing

If necessary, grind down after polymerization, then polish the restoration.

CE 0297

Does not replace the instructions for use. Please also see the Capo Bulk Fill /Capo Bulk Fill plus Composite instructions for use. The responsibility lies with the attending dentist.

SCHÜTZ DENTAL
Micerium Group

Schütz Dental GmbH

Dieselstr. 5-6 • 61191 Rosbach/Germany

Tel. +49 (0) 6003 814-365 • Fax +49 (0) 6003 814-907

www.schuetz-dental.com • export@schuetz-dental.de