# **EL-FORM S**

---

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If there is anything in this instruction for use that you do not understand, please contact our customer service department before using the product.

As the manufacturer of this medical device, we inform our users and patients that all serious events occurring in connection with it must be reported to us (the manufacturers) as well as the relevant authorities in the Member State where the user and/or patient is resident.

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## Instructions

### Indications:

- Forming crowns, inlays, onlays, posts, small bridges, and secondary crowns on plaster, metal, or plastic
- Gold-plating model casts for dentures
- Friction repair of secondary parts

### Scope of supply:

EL-FORM S Galvano gold bath rapid  
with 15 g gold / 500 ml Art. no. 625013

EL-FORM S Shine additive (5 ml)  
for gold bath 15 g Art. no. 625024

EL-FORM S Galvano gold bath standard  
with 7.5 g gold / 500 ml Art. no. 625014

EL-FORM S Shine additive (5 ml)  
for gold bath 7.5 g Art. no. 625025

EL-FORM S Pre gold plating bath  
with 1 g gold / 500 ml Art. no. 625009

EL-FORM S Activation solution  
for pre gold plating (500 ml) Art. no. 625008

**For general information on manufacturing and processing galvanized objects, please see the instructions for the EL-FORM S unit.**

### Using the EL-FORM S gold bath solution

EL-FORM S gold solution is intended exclusively for use in the EL-FORM S galvanizing unit.

The EL-FORM S galvanization unit makes it possible to precipitate galvanizing form parts by means of automatic surface calculations. After filling the complete contents of the bath (500 ml), one of three different thicknesses can be selected (200, 250 or 300 µm). Precipitation occurs automatically. Depending on consumption, the gold solution can be used for several precipitations.

For reasons of economy, the EL-FORM S gold solution should be used and filtered a maximum of 5 times. Otherwise, the gold content displayed may no longer be correct. If mainly small or only a few parts are to be galvanized or gold plated, the solution with the lower gold content (7.5 g / 500 ml) should be used.

For galvanization, the entire gold bath solution (500 ml) should be filled into the beaker. **Add all of the shine additive (5 ml) to the bath for**

**the first galvanization process.** Please ensure that there is a separate shine additive for each bath (15 g or 7.5 g gold per 500 ml). Please rinse the empty storage container with distilled water and let it drain. It will be needed later for the bath solution.

Put the **mixing rod into the beaker** and set it onto the heating element. Galvanization can now be started (for details, see the instructions for the EL-FORM S unit).

Afterwards, the gold solution can again be filled into the storage flask. Bath number and amount of residual gold should be noted on the side of the label.

The solution must be filtered before every new galvanization to remove any impurities, dust particles, etc. and to ensure that the gold plating is absolutely free of foreign bodies.

Set the enclosed filtration attachment onto the 600 ml beaker, insert the funnel, and put the filter into it. The gold solution can now be gradually filtered. Please ensure that the entire solution is filtered through.

Estimate beforehand whether the remaining amount of gold is adequate for the next process.

The gold bath solution can be used until the remaining gold is no longer adequate for another galvanization. This remaining gold can be extracted using the function „Precipitate remaining gold“ with the remaining gold sponge.

### Use of the EL-FORM S pre gold plating bath and activation solution

The EL-FORM S unit makes it possible to gold plate the bases of dentures and clasps and friction repair secondary crowns, etc. Parts made of non-precious alloys (Co-Cr or Ni based), stainless steel, and gold, **but not titanium** can be gold plated; for titanium, sufficient adhesion cannot be guaranteed.

The object must first be pre gold plated.

This requires an activation solution and a gold plating bath.

Fill all of the activation solution (500 ml) into the beaker and activate the surface of the object using current (for details, see the instructions for the EL-FORM S unit).

Afterwards, return the activation solution to the container and rinse the beaker thoroughly with water.

The activation solution can be used up to 30 times. If the solution becomes dark or fewer bubbles are formed, it should be replaced by a new solution.

The pre gold plating solution is then filled into the clean beaker, the magnetic mixer rod is attached, and the gold plating head is again set onto it with the activated and well-rinsed object. (For details, see the instructions for the EL-FORM S unit).

Here again, the pre gold plating solution can then be returned to the container. The bath can be used until the surface of the object turns gold after gold plating for a maximum of 9 minutes. Afterwards, the solution will be exhausted and must be replaced by a new gold bath solution.

Then gold plating can be carried out with the EL-FORM S gold bath (for details, see the instructions for the EL-FORM S unit).

### **Storage**

Please store the solutions at 10 - 25 °C in a dry, well-closed, dust-free container away from light. Please do not use after the expiry date or if gold flocculation is visible.

Please label solutions that have been used and do not mix with new or other bath solutions. The expiry date of the gold bath refers to the unopened flask. It is reduced after opening depending on the frequency and duration of use.

### **Important!**

Gold bath solution can cause permanent black spots on contact with skin or clothing. Wear appropriate protective clothing (lab coats) and rinse immediately with water in case of contact.

EL-FORM S pre-gilding bath is harmful to health. Please, adhere to the instructions on the packaging and on the safety data sheet.

### **Warning**

#### **EL-FORM S Vorvergoldungsbad:**

Contains: potassium tetrakis(cyano-C)aurate

#### **Hazard and precautionary statements:**

Harmful if swallowed, in contact with skin or if inhaled. Harmful to aquatic life with long lasting effects. Avoid breathing vapours/spray. Wear protective gloves/protective clothing. Rinse mouth. Take off contaminated clothing and wash it before reuse.

Composition of the gold precipitated:

Au > 99.9 %

Cu < 0.1 %

#### **Side-effects.**

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

#### **Contra-indications / interactions.**

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, we will supply the composition of our medical device upon request. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

#### **Note.**

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

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Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany