FuturaIntensiv

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Prosthetic colourant

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Colore protesi

Tinte para prótesis

SCHÜTZ  **DENTAL**
Micerium Group

Schütz Dental GmbH · Dieselstr. 5 - 6 · 61191 Rosbach/Germany
Telefon +49 (0) 6003 814-0 · Telefax +49 (0) 6003 814-906
info@schuetz-dental.de · www.schuetz-dental.de

CE 0297

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Indikation

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Processing instructions

Indication

Individual colouring of prostheses.

Processing

The Futuralintensiv colours can be processed with all denture acrylics in the Futura line.

Processing using cold / hot polymerisate

Measure out denture acrylics powder and liquid as usual. Add a small portion of Futuralintensiv to the prepared quantity of liquid and stir briefly. Now work the measured powder into the liquid as usual. Process acrylics according to the manufacturer's instructions for the denture acrylics and complete the prosthesis.

Characterisation of a finished prosthesis

Prepare the required quantities of denture acrylics powder and liquid (cold poly-

merisate). First add Futuralintensiv to the monomer liquid, then the denture acrylics powder and mix everything carefully. After a short doughing time, apply the material to the roughened surface of the prosthesis which has been wetted with monomer.

The characterised prosthesis is polymerised for 15 min. in the pressure aging vessel (2 - 4 bar, 45° C) and then finished as usual. Corrections can be made many times in the same way.

Colours

white, yellow, red, brown, blue

Only use colours in small quantities because they have a high colouring effect.

Form supplied

10 g bottle

For side effects, contra-indications and danger notes please refer to the corresponding points in the instructions for use of the denture acrylics used.

Disposal

Powder can be disposed of in household rubbish

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